




sensor

Installation guide

The PU sensor kit includes a device, a sensor plate, an inflatable, a charging plate with a holder and net adaptor, a quick start guide, and this installation guide.


Preparation of the device

1. Pull out the plastic ribbon from the device to connect the battery.
2. Screw the tube of the inflatable into the left connector **A** on the back of the device.
3. Connect the cord of the sensor plate to the right connector **B**.

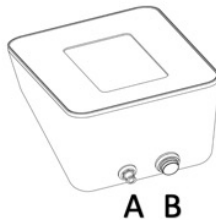


Charging

1. Connect the charging plate **A** to a mains socket and place it in the holder **B** that facilitates correct placement.
2. Place the device on the charging plate. When charging starts the lamp **C** of the charging plate changes colour from red to blue and the device emits a short beep.

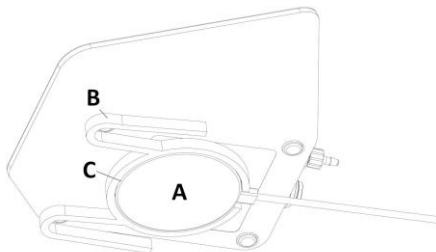



First start-up

User agreement


At the first start-up of PU sensor a user agreement is shown. Read through the entire agreement by scrolling on the screen, and accept it by pressing **Accept** .



Setting the date and time

1. Press **Info**  and then **Admin** .
2. Enter the password, which is the four last digits of the serial number, and press **Enter**. The serial number can be found at the bottom of the device.
3. Set the date and time and save by pressing **Save**.

WiFi connection

1. From the Admin page press **Network settings** to connect the device to a wireless network (WiFi).
2. Press **Add** to see the available networks.
3. Mark the network you want to connect to, enter the password of the network and press **Save**.
4. Press **Close** and then **Back**  to come back to the Admin page.



Language settings

1. From the Admin page press [Language settings](#). Find the desired language in the list and press [Select](#) to the right of the language.
2. Make sure the correct language is selected and press [Restart](#). The device will restart with the selected language.

More information: pusensor.se/en/userpages

